PP-5090-C 2019/11/08

Instructions

Single use hydrophilic urinary catheter

1. Wash your hands thoroughly.
2. Peel the packet open at the funnel end and fill with water (at home from the cold tap, in hospital sterile water or saline).
3. Remove the sticker at the top of the pack to reveal the self-adhesive patch and attach the pack to a dry surface.
4. Soak the catheter for at least 30 seconds whilst preparing yourself for catheterisation.
5. Remove the catheter from the packet.
6. Carry out the catheterisation procedure as instructed. Dispose of the catheter sensibly after use.

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www.wellspect.fr www.wellspect.ch

Instructions

Single use hydrophilic urinary catheter

Indications for use: Intermittent urinary catheterization. Before using the product make sure you receive proper training on the catheterization procedure from your healthcare provider.

1. Start by washing your hands with soap and water.
2. Open the pack carefully. Pull the tabs open about 2 inches. Pour water directly into the pack. Use sterile water only as directed by your healthcare provider.
3. The pack has a self-adhesive patch on the back. Remove the blue tab and attach the pack to a sink or other convenient, dry surface.
4. Let the catheter soak for at least 30 seconds while you are getting ready to catheterize.
5. The catheter is then ready to be inserted.
6. Carry out the catheterization procedure as instructed. After use, dispose of the catheter appropriately (local regulations may vary).

NB. LoFric® is for single use only. LoFric must not be placed on paper or cloth as these materials can absorb the catheter's liquid coating

LoFric is for prescription use only.

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