
PD-3967:F
(based on DR-3967:B)

LoFric Primo (20 cm)
191014

PMS 293 C (100 %)

C=0 M=0 Y=0 K=100 (100 %)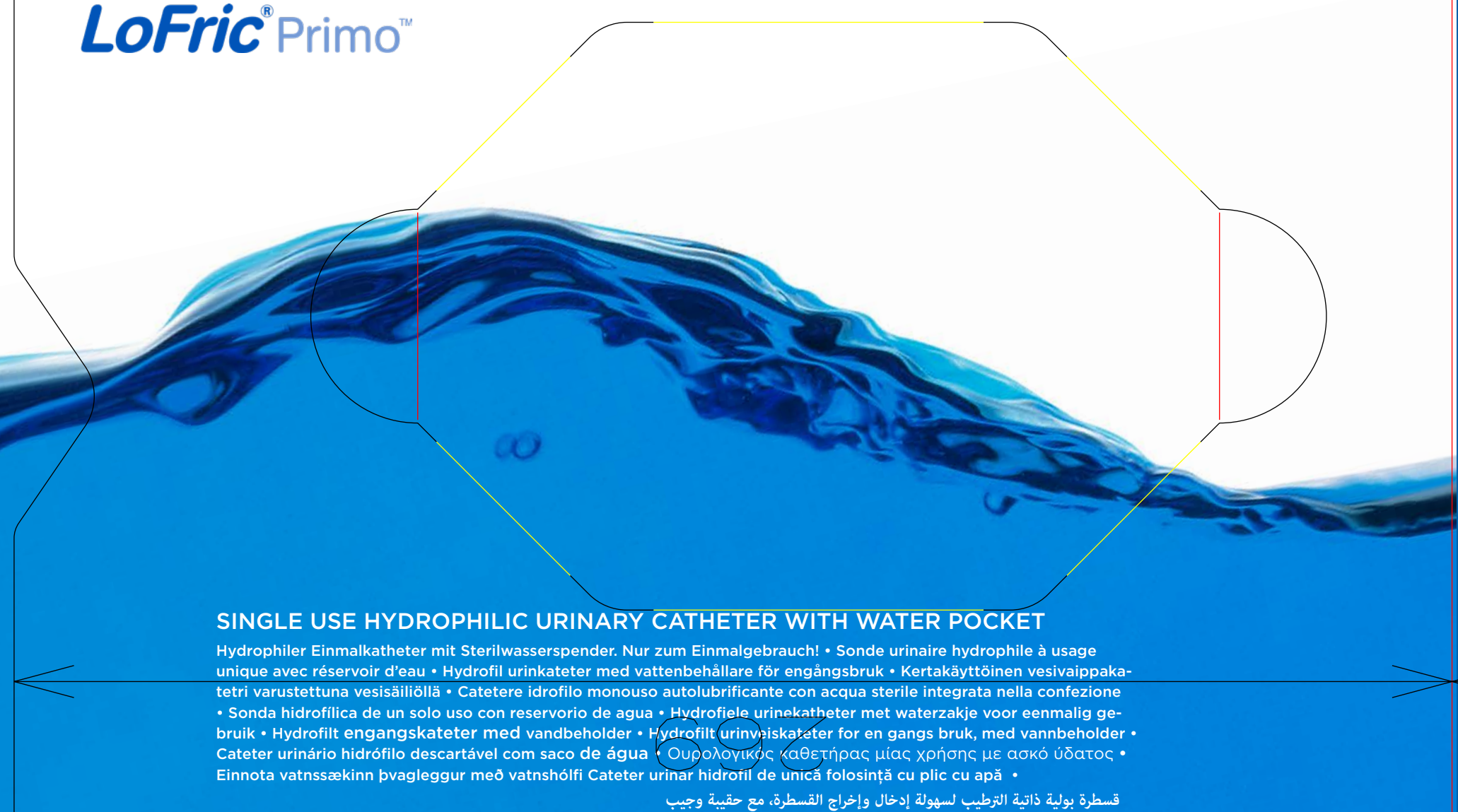

SINGLE USE HYDROPHILIC URINARY CATHETER WITH WATER POCKET

Hydrophiler Einmalkatheter mit Sterilwasserspender. Nur zum Einmalgebrauch! • Sonde urinaire hydrophile à usage unique avec réservoir d'eau • Hydrofil urinkateter med vattenbehållare för engångsbruk • Kertakäyttöinen vesivaippakattetri varustettuna vesisäiliöllä • Catetere idrofilo monouso autolubrificante con acqua sterile integrata nella confezione • Sonda hidrofílica de un solo uso con reservorio de agua • Hydrofilele urinekatheter met waterzakje voor eenmalig gebruik • Hydrofílt engangskateter med vandbeholder • Hydrofíltt urínvaskateter for en gangs bruk, med vannbeholder • Cateter urinário hidrófilo descartável com saco de água • Ουρολογικός καθετήρας μίας χρήσης με ασκό ύδατος • Einnota vatnssækinn þvagleggur með vatnshólfí Cateter urinar hidrófil de unică folosință cu plic cu apă •

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Instructions US/UK/AUS
Indications for use: Intermittent urinary catheterization.

Before using the product make sure you receive proper training on the catheterization procedure from your healthcare provider. Start by washing your hands with soap and water.

- Unfold the package. Hold the product upright.
- Fold the water pocket.
- Squeeze the water pocket and let the water run down to the catheter.
- Soak the catheter for 30 seconds while you are getting ready to catheterize.
- Open the product. Catheterize.

The water pocket contains sterile water that should only be used for soaking the LoFric catheter.

After use, dispose of the catheter appropriately (state and local regulations may vary).

NB. LoFric Primo is for single use only.

LoFric Primo must not be placed on paper or cloth as these materials can absorb the catheter's liquid

For USA: LoFric Primo is for prescription use only.

When sold in Australia sponsored by: Dentsply Sirona Pty Ltd, 11-21 Gilby Road, Mount Waverley VIC 3149

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